



C-Pette™

Step 1

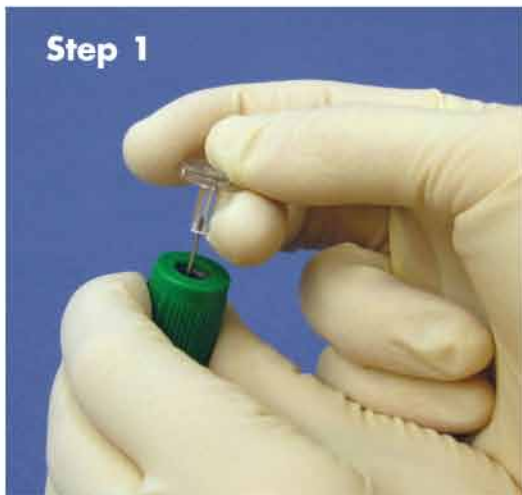

The C-Pette™ makes delivering sample into test cards and point of care cassettes simple and efficient. Since you don't have to open the primary sample tube any risk of exposure to bio-hazards is reduced. C-Pette™ is ideal for use with many point of care analyzers such as the Abbott® i-Stat®. You can use the C-Pette™ with any analyzer that requires the delivery of a sample to a test cartridge.

Step 2

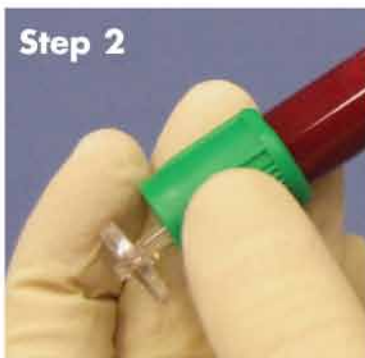

Step 3



Step 4



Using C-Pette

Step 1: Grasp the primary blood collection tube by the plastic skirt covering the rubber stopper and insert the metal cannula thru the stopper.

Step 2: Re-suspend cells just prior to dispensing by holding the tube by the plastic skirt and inverting gently as needed.

Step 3: Place the C-Pette dropper tip into the cassette loading port.

Step 4: Push down and release to completely fill cassette reservoir. If a larger sample volume is required, repeat push and release of tube.

Cat. No.
5-7630-500-000

Item
C-Pette for Cartridges, 5 boxes of 100

i-STAT is a registered trademark of the Abbott Group of Companies

Helena Plastics
www.helenaplastics.com
800-227-1727
Made in U.S.A.